



Installation Guide

EVACU B Oxygen Supply Manifold


2017



Step 1: Install O2 Hose to Oxygen Supply Manifold

Attach Green Oxygen (O2) Hose to the end of the Oxygen Supply Manifold furthest away from the Pressure Gauge.

Ex:

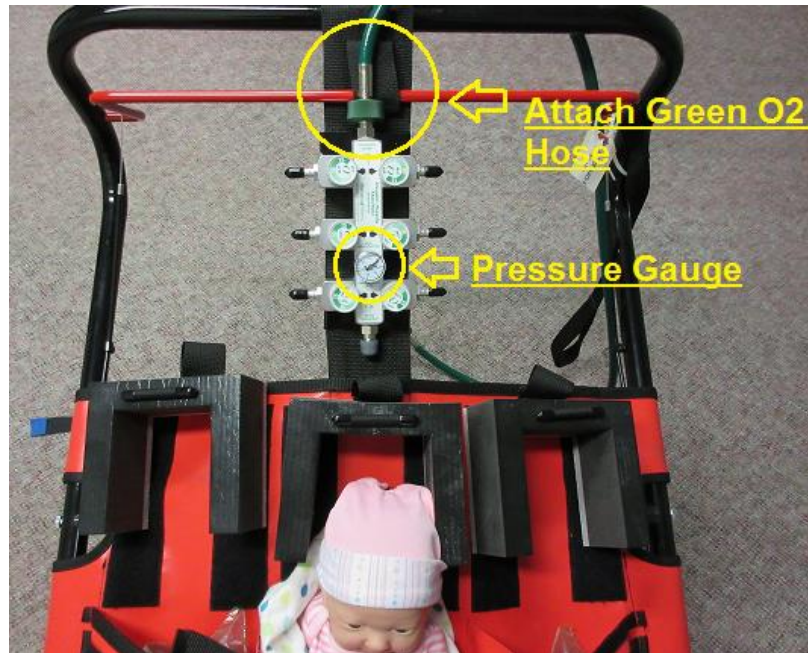

Step 2: Securing the Oxygen Supply Manifold to the EVACU B

- Position the Oxygen Supply Manifold so that the hose comes towards the brake handle and the pressure gauge is readable to the user of the EVACU B
- Secure the two top back clips of Oxygen Supply Manifold inside the loop of the centre apron strap (under brake handle). *****This ensures that the Oxygen Supply Manifold will not slide down the centre strap.*****
- Next secure the bottom two back clips around the apron strap (there is no loop). Installation is now complete

Two Picture Examples on Following Page



DORO INC.

Safety & Security Products

